
Modelo: iWear Extraclean Multi REF Troquel: TRO-PROS-014 Dimensiones: 540 x 175 mm	Colores: black
Código: PROS588	

175 mm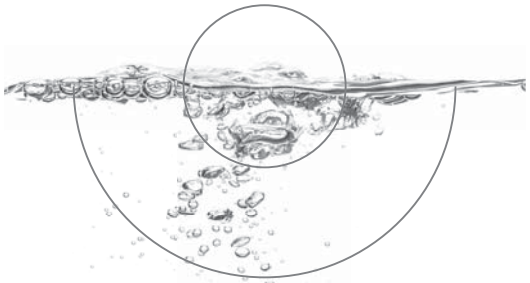

extraclean
all-in-1

gb ALL IN ONE SOLUTION FOR USERS WITH SENSITIVE EYES. FOR ALL TYPES OF SOFT CONTACT LENSES (including the new generation of silicone hydrogel-lenses).

iWear® Extraclean has been especially formulated to provide extra comfort to even the most sensitive contact lens users.

Funcions: Cleans and eliminates proteins, disinfects, stores, rinses, wets and lubricates.

INSTRUCTIONS FOR USE:

- Before handling your lenses, wash your hands with soap and water.
- Place your lens on the palm of your hand and rub it carefully with a few drops of iWear® Extraclean solution, on both sides for a few seconds.
- Rinse your lenses with iWear® Extraclean solution.
- Place your lenses in the lens case, and be sure to cover these with iWear® Extraclean solution.
- Keep lenses at least 4 hours in iWear® Extraclean solution (or overnight) to grant an adequate disinfection.
- Once the disinfection process is done, you may insert your lenses directly to your eye or you may rinse them previously with iWear® Extraclean solution.
- Keep your lens case clean and be sure to replace the iWear® Extraclean solution in your lens case daily. Do not use tap water to clean the lens case.

Last revision date: 2009/11

PRECAUTIONS:

- Once opened, the bottle should not be used after 90 days.
- Maintain the bottle well closed after use.
- Do not use the product if the safety seal is missing or broken.
- You may store your lenses up to 30 days in the closed lens case.
- Do not ingest the solution and keep away from children's reach.
- Keep at room temperature.
- Do not allow the dropper to come into contact with any surface to avoid contamination.
- In case of irritation, consult your contact lens specialist.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.
- Do not use if you are allergic to any of the components.
- Never use tap water to rinse your lenses or lens case.
- Do not use after the expiration date.
- Discard the solution in the lens case after each use.

COMPOSITION:
Sodium Hyaluronate, Poloxamer, EDTA, Polyhexanide 0,0001% in a buffered, isotonic and sterile solution.

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GrandVision Supply Chain B.V.
AVIZOR S.A.